

CLINICAL CHEMISTRY | HEMOSTASIS | INFECTIOUS DISEASE | POINT OF CARE

# Uni-Gold™ Recombigen® HIV-1/2



## Third Generation Rapid HIV-1/2 Test Results in 10 Minutes

### BENEFITS

Uni-Gold™ Recombigen® HIV-1/2 is a highly reliable test which provides these important benefits:

- CLIA-Waived (whole blood or finger stick)
- Moderately Complex (serum or plasma)
- Third generation technology
- One year expiration from date of manufacturing



**THREE SIMPLE STEPS:**

Add one drop of patient sample



Add four drops of wash solution



Read results in only 10 minutes



**INTERPRETATION:**

Positive Result



Negative Result



**PERFORMANCE CHARACTERISTICS:**

Sensitivity: HIV-1 100%; HIV-2 99.3%

Specificity: 99.7%

For full instructions, please refer to the package insert.

**ORDERING INFORMATION:**

Product	Part Number	Quantity	Storage
Uni-Gold™ Recombigen® HIV-1/2	1206506	20 Tests	2-27°C
Uni-Gold™ Recombigen® HIV-1/2 Controls	1206530	0.5 ml HIV 1 positive 0.5 ml HIV 2 positive 0.5 ml negative	2-8°C

CPT CODE: 86703



CLIA-Waived (whole blood)  
Fingerstick-Approved

Distributed by:



LifeSign LLC  
85 Orchard Road  
Skillman, NJ 08558

Tel: 800-526-2125  
info@lifesignedmed.com  
www.lifesignedmed.com



Trinity Biotech USA  
2823 Girls Road  
Jamestown, NY 14701

Tel: 800-325-3424  
info@trinitybiotech.com  
www.trinitybiotech.com