



Status™

lifeSign

COVID-19/Flu A&B

A rapid immunoassay for the simultaneous direct detection and differential diagnosis of SARS-CoV-2, Influenza Type A and Type B antigens from nasal and nasopharyngeal swab specimens

Infections with these viruses may present similar symptoms. Can you tell them apart? **We Can.**

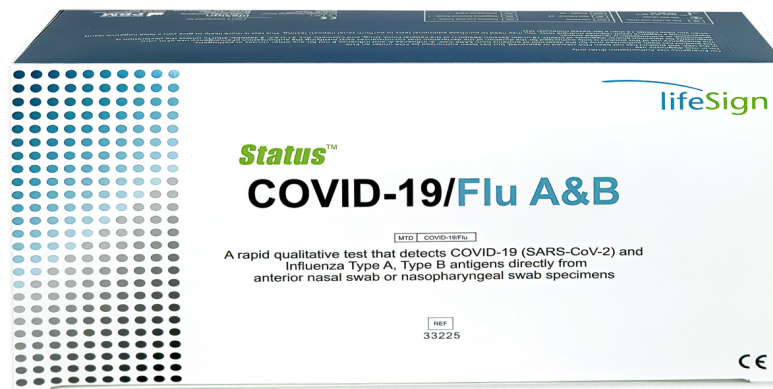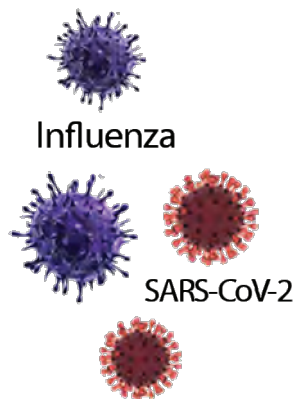

Learn More

1 Test + 1 Specimen = 3 Results

The additive power of **Status™ COVID-19/Flu A&B**

- **Sensitivity and Specificity**

- CLIA Waived for use with Anterior Nasal and Nasopharyngeal swab specimens
- COVID-19 -Anterior nasal - PPA 97.4% , NPA 100%
Nasopharyngeal -PPA 95.5%, NPA 99.7%
- FLU A - Anterior nasal - PPA 91.5%, NPA 99.5%.
Nasopharyngeal - PPA 94.1%, NPA 99.4%.
- FLU B - Anterior nasal - PPA 90.2%, NPA 99.8%.
Nasopharyngeal - PPA 92.7%, NPA 100%

- 28 month shelf-life from date of manufacture
- Visually read in 15 minutes

ORDERING INFORMATION		
PRODUCT	# OF TESTS	ITEM #
Status™ Covid-19/Flu A&B	25	33225


 Proudly made in the USA

For more information or to place an order, contact:


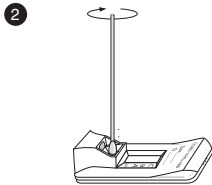
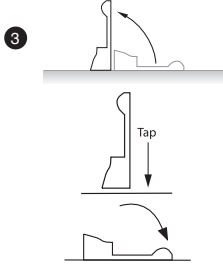
lifeSign

A PBM Group Company

85 Orchard Road • Skillman • New Jersey • 08558
800.526.2125 • 732. 246. 3366 • Fax: 732.246.0570 • www.lifesignmed.com

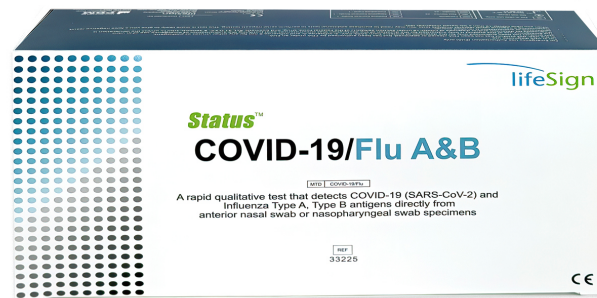

Easy 3 step procedure

	<p>1 squeeze</p> <p>Tear the tab off the extraction reagent capsule and dispense entire contents into the extraction well.</p>
	<p>2</p> <p>Insert the specimen swab in the swab stand.</p> <ul style="list-style-type: none"> • Rotate swab 3 times to mix the specimen. • Let Stand 1 minute. • Rotate swab 3 times again and discard the swab.
	<p>3</p> <p>Raise the device upright and let stand 1-2 seconds.</p> <p>Gently tap the device to ensure the liquid flows into the hole.</p> <p>Lay the device back down.</p>
<p>Set a timer for 15 minutes.</p>	
<p>4</p>	<p>Read test results at 15 minutes.</p> <p>NOTE: False positive or false negative results can occur if the test is not read between 15 and 20 minutes.</p>

- Refer to our website at www.lifesignmed.com for complete test procedure details, training video and package insert.
- **Status™ COVID-19/FLU A&B** is an antigen test for use with anterior nasal and nasopharyngeal swab specimens.
- **Status™ COVID-19/FLU A&B** has been granted FDA 510K Clearance

 **Proudly made in the USA**

For more information or to place an order, contact:



A PBM Group Company

85 Orchard Road • Skillman • New Jersey • 08558
800.526.2125 • 732. 246. 3366 • Fax: 732.246.0570 • www.lifesignmed.com

Printed in USA
All rights reserved
83488 - 12/12/2025